

clinell®

www.clinell.com

EFFECTIVE IN
AS LITTLE AS
10 SECONDS



Universal Wipes

FOR CLEANING & DISINFECTING SURFACES,
HANDS AND EQUIPMENT IN A SINGLE STEP

- ✓ Multi-purpose cleaning and disinfection solution trusted by the NHS
- ✓ Large and strong wipes kill at least 99.99% of bacteria and viruses
- ✓ Suitable for use on hands and a wide range of hard surfaces without damage
- ✓ Thick, heavy-duty spunlace wipes, suitable for frequent use

Antimicrobial wipes for cleaning & disinfecting surfaces, hands and equipment



Kills at least 99.99% of bacteria and viruses including Coronavirus.

- ✓ Multipurpose cleaning and disinfecting solution
- ✓ Dermatologically tested - contains aloe vera and moisturisers to be gentle on skin
- ✓ Effective in 10 seconds
- ✓ Kills at least 99.99% of bacteria and viruses
- ✓ Can be used on visibly dirty hands, unlike alcohol gels
- ✓ Suitable for use on a wide range of hard surfaces including: Car/van interiors, tablets, mobile phones, door handles and remote controls to name but a few.
- ✓ Designed by doctors. Trusted by healthcare professionals



ON HANDS



OUT DRIVING



TABLETS/PHONES



WORK/OFFICE



WHEN UNWELL



ON THE GO

INSTRUCTIONS FOR USE ON HANDS

Wipe hands thoroughly for 60 seconds.

1. Rub hands palm to palm.
2. Rub back of each hand with the palm of other hand, fingers interlaced.
3. Rub palm to palm with fingers interlaced.
4. Rub with backs of fingers to opposing palms with fingers interlocked.
5. Rub each thumb clasped in opposite hand using rotational movement.
6. Rub tips of fingers in opposite palm in a circular motion.
7. Rub each wrist with opposite hand.
8. Discard the wipe in the appropriate waste bin.

INSTRUCTIONS FOR USE ON SURFACES

1. Remove one wipe from the pack.
2. Wipe in an 'S' shaped pattern from clean to dirty.
3. Discard the wipe in the appropriate waste bin.
4. Allow to air dry.



KITCHENS



BATHROOMS

BACTERIA	CONTACT TIME
<i>Staphylococcus aureus</i> (MRSA)	10 sec
<i>Klebsiella pneumoniae</i> (ESBL)	10 sec
<i>Staphylococcus aureus</i>	10 sec
<i>Escherichia coli</i>	10 sec
<i>Listeria monocytogenes</i>	10 sec
<i>Campylobacter jejuni</i>	10 sec
<i>Pseudomonas aeruginosa</i>	10 sec
<i>Salmonella typhimurium</i>	10 sec
FUNGI	
<i>Candida albicans</i>	10 sec
<i>Candida auris</i>	10 sec
VIRUSES	
Adenovirus	30 sec
Norovirus	30 sec
Human Coronavirus	60 sec

PRODUCT	UNIT OF ISSUE	NON-WOVEN	WIPE SIZE	LID TYPE	BIOCIDE PRODUCT TYPE	PRODUCT CODE
Universal Wipes 100	Pack of 100	Spunlace 60gsm	26 x 18cm	Plastic lid	PT1 & PT2	BCW100

Use biocides safely. Always read the label and product information before use.

JBN200182