

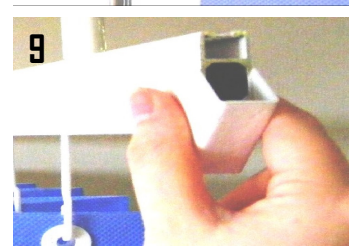
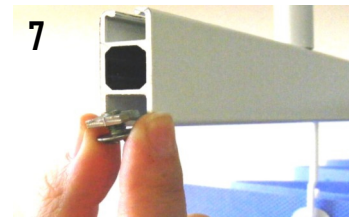
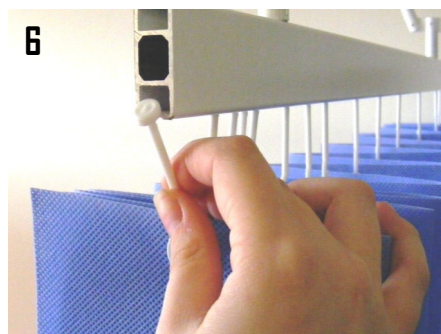
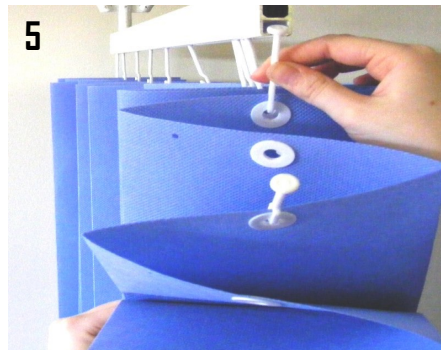


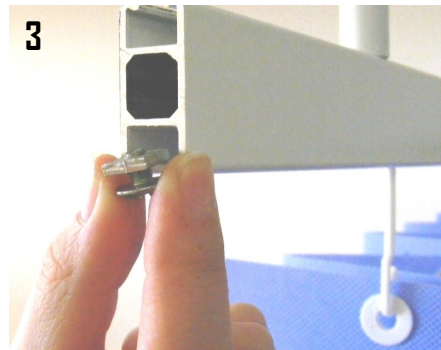

MARLUX MEDICAL

UNI-GLIDE DISPOSABLE PRIVACY CURTAIN INSTRUCTIONS FOR USE

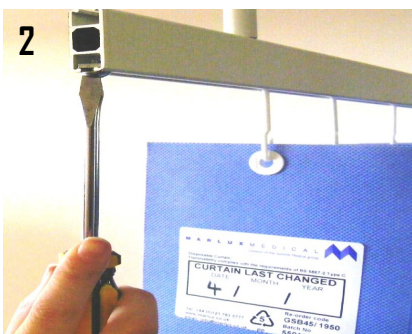

- 1 Remove the track cover, if present, on the end of the curtain track.
- 2 Loosen the end stop with the aid of a screwdriver
- 3 Remove the end stop

- 4 Remove the existing curtain and if necessary the gliders.
All Marlux[®] curtains are 100% recyclable. Marlux[®] offers a recycling service to its customers (subject to conditions). Please contact Marlux[®] for more information.



- 5 Open the plastic sleeve at the hook end of the new disposable curtain.
- 6 Do not fully remove the plastic sleeve at this stage. Simply slide the pre-fitted uni-glide hooks onto the existing curtain track.

- 7 Replace the end stop.
- 8 Tighten the end stop with the aid of a screwdriver.
- 9 If present, replace the track cover on the end of the curtain track

- 10 Remove the plastic sleeve completely. Write the curtain installation date in the area provided.

To change the curtain repeat from step 1 with a new Marlux[®] curtain.