




M A R L U X M E D I C A L

UNIVERSAL DISPOSABLE PRIVACY CURTAIN INSTRUCTIONS FOR USE

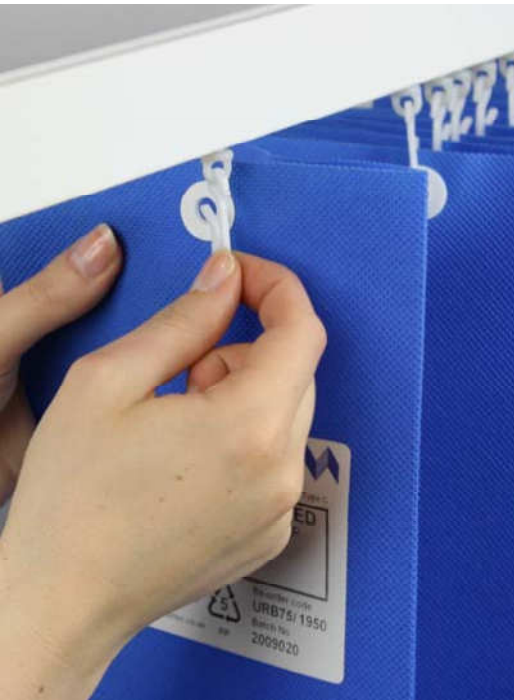


1 Open the plastic sleeve at the top of the disposable curtain. Do not fully remove the plastic sleeve at this stage. Hook every second reinforced eyelet on to the existing curtain tracks.

2 Remove the plastic sleeve completely. The curtains have a maximum drop of 1950mm but may be cut to a shorter length.

3 Write the curtain installation date in the area provided. This specially designed anti-peel, durable labelling enables self-auditing of individual curtain change policies.

4 To change the curtain simply un-hook the eyelets, then repeat from step 1 with a new Marlux® curtain.

 All Marlux® curtains are 100% recyclable. Marlux® offers a recycling service to its customers (subject to conditions); which includes the collection and ethical recycling of used product.

For more information regarding the Enviro Bag service please contact Marlux®

www.marlux.co.uk Tel: +44 (0) 121 783 5777 Email: sales@marlux.co.uk